



25Pcs.

IV Infusion Set



INSTRUCTIONS FOR USE

MATERIALS USED

Polypropylene, ABS, PVC, Nylon, Latex Rubber or Poly Isoprene, Stainless Steel, Poly Carbonate, PE.

INDICATIONS

Infusion of IV fluids, parenteral nutrition and administration of other drugs & medicines.
Maintaining hydration in patients who are unable to take sufficient volume of oral fluids.

CONTRAINDICATIONS

Administration of highly viscous fluids.
Blood Transfusion.
Not to be used in patients with known hypersensitivity to any of the materials used.

DIRECTIONS FOR USE

1. Close the Regulating Clamp.
2. Remove the protective cap from the spike and insert the spike into the I.V. fluid container to its full length.
3. Open the airvent cap of piercing spike.
4. Squeeze the drip chamber and adjust the fluid level till half filled.
5. Open regulating clamp and allow solution to pass until air bubbles in the tube are removed completely, close the regulating clamp.
6. Connect the IV set with I.V. device and regulate the flow by adjusting regulating clamp to achieve desired flow rate.

NOTE : Point No. 3 is applicable only for I.V. Set with airvent.



CAUTIONS

- Do not use if protective cap is loose or missing.
- Do not use if package is opened or damaged.
- Use rubber part or Y site only for injecting the medicines using aseptic techniques.
- Close the airvent during periods of interrupted infusion therapy (applicable only for I.V. Set with airvent).
- For gravity feed only.
- The product should be used only by qualified healthcare professionals.
- Do not re-sterilize. Discard the set after single use.
- Reuse and cleaning of product may alter their structural and mechanical properties. It may lead to infection or other illness/injury.
- The product should be replaced and disposed of as per facility approved protocol or CDC guidelines.
- Store at room temperature, avoid excessive heat, protect from direct sunlight and moisture.
- Poly Medicure Limited will not be responsible for any direct incidental or consequential damages resulting from reuse of product.



•NON-TOXIC
•NON-PYROGENIC

EU REP Obelis s.a
Boulevard Général Wahis 53
1030 Brussels, BELGIUM
Tel : +(32) 2. 732.59.54
E-mail : mail@obelis.net



Web : www.polymedicure.com
Customer care no. : +91-129-4287053,
Email: customercare@polymedicure.com,

Rev. 08-25/08