

# INSTRUCTIONS FOR USE

## Polyfusion Plus

### MATERIALS USED

- Polypropylene, Acrylonitrile Butadiene Styrene, Polyvinyl Chloride, Nylon, Latex Rubber or Poly Isoprene, Stainless Steel, Poly Carbonate, Polyethylene.

### INDICATIONS

- Infusion of IV fluids, parenteral nutrition and administration of other drugs & medicines.
- Maintaining hydration in patients who are unable to take sufficient volume of oral fluids.

### CONTRAINDICATIONS

- Administration of highly viscous fluids.
- Blood Transfusion.
- Not to be used in patients with known hypersensitivity to any of the materials used.

### DIRECTIONS FOR USE

1. Close the Regulating Clamp.
2. Remove the protective cap from the spike and insert the spike into the I.V. fluid container to its full length.
3. Open the airvent cap of piercing spike.
4. Squeeze the drip chamber and adjust the fluid level till half filled.
5. Open regulating clamp and allow solution to pass until air bubbles in the tube are removed completely, close the regulating clamp.
6. Connect the IV set with I.V. device and regulate the flow by adjusting regulating clamp to achieve desired flow rate.

NOTE : Point No. 3 is applicable only for I.V. Set with airvent.

### CAUTIONS

- Do not use if protective cap is loose or missing.
- Do not use if package is opened or damaged.
- Use rubber part or Y site only for injecting the medicines using aseptic techniques.
- Close the airvent during periods of interrupted infusion therapy (applicable only for I.V. Set with airvent).
- For gravity feed only.
- The product should be used only by qualified healthcare professionals.
- Do not re-sterilize. Discard the set after single use.
- Reuse and cleaning of product may alter their structural and mechanical properties. It may lead to infection or other illness/injury.
- The product should be replaced and disposed of as per facility approved protocol or CDC guidelines.
- Store in between 5°C to 35°C, avoid excessive heat, protect from direct sunlight and moisture.
- Poly Medure Limited will not be responsible for any direct incidental or consequential damages resulting from reuse of product.



### POLY MEDICURE LTD.

Plot No. 17, Sector-3, IIE, SIDCUL Haridwar,  
Uttarakhand 249403, INDIA  
Web : [www.polymedicure.com](http://www.polymedicure.com)  
Contact customer care executive: ☎+91-129-4287053,  
email: [customer care@polymedicure.com](mailto:customer care@polymedicure.com)  
Address: Same as above

**EU REP** Obelis s.a.,  
Boulevard Général Wahis 53,  
1030, Brussels, BELGIUM.  
Tel : +(32) 2. 732.59.54  
E-mail : [mail@obelis.net](mailto:mail@obelis.net)



Cautions



See Instructions for use



Product reference/Art. No.



For single use only



Do not Resterilize



Do not use if packaging or product has been damaged or contaminated



Batch Number



Date of Manufacturing



Use by / Expiry date



Single Sterile Barrier System



Sterilised by Ethylene Oxide



European Authorised Representative



Manufactured by



Non Pyrogenic



Storage Condition



Keep Away from Sunlight



Keep Dry



Medical Devices



Not made with DEHP plasticizers