

INSTRUCTIONS FOR USE

IV Infusion Set with Airvent & Priming Filter **ULTRAFUSION**

Materials Used

- Polypropylene, Acrylonitrile Butadiene Styrene (ABS), PVC, Poly Carbonate.

Indications

- Infusion of multiple IV fluids, administration of other drugs and medicines.
- Maintaining hydration in patients who are unable to take sufficient volume of oral fluids.

Contraindications

- Administration of highly viscous fluids.
- Blood Transfusion.
- Not to be used in patients with known hypersensitivity to any of the materials used.

Directions for Use

- Close the Regulating Clamp.
- Remove the protective cap from the spike and insert the spike into the I.V. fluid container to its full length.
- Squeeze the drip chamber and adjust the fluid level till half filled.
- Open the airvent cap of piercing spike.
- Open regulating clamp and priming takes place automatically up to integrated filter (maintains a close system thus prevents contamination). Close the regulating clamp.
- Remove the priming filter cap.
- Connect the IV set with I.V. device and regulate the flow by adjusting regulating clamp to achieve desired flow rate.

Cautions

- Do not use if protective cap is loose or missing.
- Do not use if package is opened or damaged.
- For pressure and gravity feed only.
- Use Y site only for injecting the medicines using aseptic techniques.
- Close the airvent during periods of interrupted infusion therapy.
- Priming filter for single use only.
- The product should be used only by qualified healthcare professionals.
- Do not re-sterilize. Discard the set after single use.
- Reuse and cleaning of product may alter their structural and mechanical properties. It may lead to infection or other illness/injury.
- The product should be replaced and disposed of as per facility approved protocol or CDC guidelines.
- Store in between 5°C to 35°C, avoid excessive heat, protect from direct sunlight and moisture.
- Poly Medisure Limited will not be responsible for any direct incidental or consequential damages resulting from reuse of product.



POLY MEDICURE LTD.

Plot No. 17, Sector-3, Integrated Industrial Estate,
SIDCUL, Haridwar, Uttarakhand 249403, INDIA,

Web : www.polymedicure.com

Contact customer care executive: ☎ +91-129-4287053,

Email: customer care@polymedicure.com

Address: Same as above

Not made with natural rubber latex
Not made with DEHP plasticizers



Cautions



See Instructions for use



Product reference/Art. No.



For single use only



Do not Resterilize



Do not use if packaging or product has been damaged or contaminated



Batch Number



Date of Manufacturing



Use by / Expiry date



Sterilised by Ethylene Oxide



Single Sterile Barrier System



Manufactured by



Non Pyrogenic



Storage Condition



Keep Away from Sunlight



Keep Dry



Medical Devices