

# INSTRUCTIONS FOR USE

## GUEDEL AIRWAY

### Materials Used

- Acrylonitrile Butadiene Styrene (ABS), LDPE.

### Indications

- To keep oropharyngeal airway unobstructed.

### Contraindications

- Not to be used in patients with known hypersensitivity to any of the materials used.
- Not to be used on any patient having an active gag reflex or known or suspected palate fractures.

### Directions for Use

- Select an airway and measure from the level of the teeth to the angle of the jaw.
- Open the patient's mouth by using the crossed-finger or scissor technique.
- Keep the patient in lying position and place the guedel airway into the mouth between upper and lower jaws.
- Insert the guedel airway upside down and when it touches the roof of the back of the mouth, rotate it 180 degrees while continuing to advance the airway until the flange is resting on the patient's front teeth.
- If patient begins to retch/gag, remove the airway.
- The guedel airway is removed by simply pulling it out of the patient's mouth following the natural anatomical curve of the airway and the curve of the device.
- Do not rotate the airway when removing it.

### Cautions


- Do not use if package is opened or damaged.
- The product should be used only by qualified healthcare professionals.
- Do not re-sterilize. Discard the set after single use.
- Reuse and cleaning of product may alter their structural and mechanical properties. It may lead to infection or other illness/injury.
- The product should be replaced and disposed of as per facility approved protocol or CDC guidelines.
- Store in between 5°C to 35°C, avoid excessive heat, protect from direct sunlight and moisture.
- Poly Medicure Limited will not be responsible for any direct incidental or consequential damages resulting from reuse of product.

 POLYMED

Reg No. under Rule-13(2) of PWM Rules:  
BO-04-000-12-AAACP3891P-24

 POLY MEDICURE LTD.

At: A-22 TO A-27, Bais Godam, Industrial Estate,  
Jaipur, Rajasthan 302006, India  
Web : [www.polymedicure.com](http://www.polymedicure.com)  
Contact customer care executive:  
☎ +91-129-4287053.  
Email: [customercare@polymedicure.com](mailto:customercare@polymedicure.com)  
Address: Same as above

 Obelis S.a  
Boulevard Général Wahis 53  
1030 Brussels, BELGIUM  
Tel : +(32) 2. 732.59.54  
E-mail : [mail@obelis.net](mailto:mail@obelis.net)

 CE  
0123



Cautions



See Instructions for use



Product reference/Art. No.



For single use only



Do not Resterilize



Do not use if packaging or product has been damaged or contaminated



Batch Number



Date of Manufacturing



Use by / Expiry date



Sterilised by Ethylene Oxide



Single Sterile Barrier System



European Authorised Representative



Manufactured by



Non Pyrogenic



Storage Condition



Keep Away from Sunlight



Keep Dry



Medical Devices

Not made with natural rubber latex  
Not made with DEHP plasticizers