

INSTRUCTIONS FOR USE

NELATON CATHETER

Materials Used

- Poly Vinyl Chloride (PVC).

Indications

- To drain the bladder in case of urinary retention.
- To empty the bladder prior to surgery/investigations.

Contraindications

- Not to be used in patients with known hypersensitivity to any of the materials used.

Directions for Use

- Clean the genitalia of the patient with mild antiseptic solution.
- Put some antiseptic jelly on the tip of the catheter without touching the tip of the catheter.
- Advance the catheter into the urethra to its full length to the bladder.
- Once the catheter reaches the bladder the urine starts to come out. Collect the urine into the urine pot.

Cautions

- Do not use if package is opened or damaged.
- Aseptic techniques must be followed during insertion of catheter.
- The product should be used only by qualified healthcare professionals.
- Do not re-sterilize. Discard the set after single use.
- Reuse and cleaning of product may alter their structural and mechanical properties. It may lead to infection or other illness/injury.
- The product should be replaced and disposed of as per facility approved protocol or CDC guidelines.
- Store in between 5°C to 35°C, avoid excessive heat, protect from direct sunlight and moisture.
- Poly Medicure Limited will not be responsible for any direct incidental or consequential damages resulting from reuse of product.



Cautions



See Instructions for use



Product reference/Art. No.



For single use only



Do not Resterilize



Do not use if packaging or product has been damaged or contaminated



Batch Number



Date of Manufacturing



Use by / Expiry date



Sterilised by Ethylene Oxide



Single Sterile Barrier System



European Authorised Representative



Manufactured by



Non Pyrogenic



Storage Condition



Keep Away from Sunlight



Keep Dry




Medical Devices

 **POLYMED**

Reg No. under Rule-13(2) of PWM Rules:
BO-04-000-12-AAACP3891P-24

 **POLY MEDICURE LTD.**

At: A-22 TO A-27, Bais Godam, Industrial Estate,
Jaipur, Rajasthan 302006, India
Web : www.polymedicure.com
Contact customer care executive:
☎ +91 -129-4287053,
Email:customercare@polymedicure.com
Address: Same as above

 Obelis S.a
Boulevard Général Wahis 53
1030 Brussels, BELGIUM
Tel : +(32) 2. 732.59.54
E-mail : mail@obelis.net


0123

Not made with natural rubber latex
Not made with DEHP plasticizers

Rev. 07-25/08