

INSTRUCTIONS FOR USE

OXYGEN MASK

Materials Used

- Poly Vinyl Chloride (PVC), Polypropylene, Elastic Rubber.

Indications

- Administration of oxygen.

Contraindications

- Not be used in patients with known hypersensitivity to any of materials used.

Directions for Use

- Attach the oxygen supply tubing to the oxygen source and set the oxygen to the prescribed flow.
- Check for oxygen flow through the device.
- Place the mask on the patient's face with the elastic strap below the ear and around the neck.
- Gently pull the end of the strap to ensure a comfortable fit.

Cautions

- Ensure that all connections are secure.
- The tubing must only be used for connection between a oxygen flowmeter and an oxygen therapy or aerosol therapy device. Do not connect this tubing to a luer connector.
- Oxygen concentration is affected by patient's tidal volume. To avoid rebreathing exhaled CO₂, the oxygen flow rate should be at least 5L/min.
- The product should be used only by qualified healthcare professionals.
- Single patient use only.
- Discard the set after single patient use.
- Reuse and cleaning of product may alter their structural and mechanical properties. It may lead to infection or other illness/injury.
- The product should be replaced and disposed of as per facility approved protocol or CDC guidelines.
- Store in between 5°C to 35°C, avoid excessive heat, protect from direct sunlight and moisture.
- Poly Medicure Limited will not be responsible for any direct incidental or consequential damages resulting from reuse of product.



Cautions



See Instructions for use



Product reference/Art. No.



For single use only



Do not use if packaging or product has been damaged or contaminated



Batch Number



Date of Manufacturing



Use by / Expiry date



Non Sterile



European Authorised Representative



Manufactured by



Storage Condition



Keep Away from Sunlight



Keep Dry




Medical Device

 **POLYMED**

 **POLY MEDICURE LTD.**

Plot No. 104-105, Sector-59, HSIDC Industrial Area,
Ballabhgarh Faridabad, Haryana-121004, INDIA
Customer Care No. : +91-129-4287053
E-mail : customercare@polymedicure.com
Web : www.polymedicure.com

 Obelis S.a
Boulevard Général Wahis 53
1030 Brussels, BELGIUM
Tel : +(32) 2. 732.59.54
E-mail : mail@obelis.net



Not made with natural rubber latex
Not made with DEHP plasticizers