

INSTRUCTIONS FOR USE

PHOTOFUSION SET I.V. Infusion Set (Vented)

MATERIALS USED :

- Polypropylene, Acrylonitrile Butadiene Styrene (ABS), Poly Vinyl Chloride (PVC)

INDICATIONS :

- Infusion of photo sensitive drugs .

CONTRAINDICATIONS :

- Administration of highly viscous fluids.
- Blood Transfusion.
- Not to be used in patients with known hypersensitivity to any of the materials used.

DIRECTIONS FOR USE :

- Close the Regulating Clamp.
- Remove the protective cap from the spike and insert the spike into the I.V. fluid container to its full length.
- Squeeze the drip chamber and adjust the fluid level about 2/3rd filled.
- Open the air vent cap of piercing spike.
- Open the regulating clamp, priming takes place automatically up to integrated filter (maintains a close system thus prevents contamination). Close the regulating clamp.
- Remove the priming filter cap.
- Connect the IV set with I.V. device and regulate the flow by adjusting regulating clamp to achieve desired flow rate.

CAUTIONS :

- Do not use if protective cap is loose or missing.
- Do not use if package is opened or damaged.
- For gravity feed only.
- Close the airvent during periods of interrupted infusion therapy.
- The product should be used only by qualified healthcare professionals.
- Do not re-sterilize. Discard the set after single use.
- Reuse and cleaning of product may alter their structural and mechanical properties. It may lead to infection or other illness/injury.
- The product should be replaced and disposed of as per facility approved protocol or CDC guidelines.
- Store in between 5°C to 35°C, avoid excessive heat, protect from direct sunlight and moisture.
- Poly Medcure Limited will not be responsible for any direct incidental or consequential damages resulting from reuse of product.



Reg No. under Rule-13(2) of PWM Rules:
BO-04-000-12-AAACP3891P-24

POLY MEDCURE LTD.

At: A-22 TO A-27, Bais Godam, Industrial Estate,
Jaipur, Rajasthan 302006, India
Web : www.polymedcure.com
Contact customer care executive: ☎ +91-129-4287053,
Email: customercare@polymedcure.com
Address: Same as above

EU REP Obelis s.a.,
Boulevard Général Wahis 53,
1030, Brussels, BELGIUM.
Tel : +(32) 2. 732.59.54
E-mail : mail@obelis.net



Not made with DEHP plasticizer



Cautions



See Instructions for use



Product reference/Art. No.



For single use only



Do not Resterilize



Do not use if packaging or product has been damaged or contaminated



Batch Number



Date of Manufacturing



Use by / Expiry date



Sterilised by Ethylene Oxide



Single Sterile Barrier System



European Authorised Representative



Manufactured by



Non-Pyrogenic



Storage Condition



Keep Away from Sunlight



Keep Dry



Medical Devices