

INSTRUCTIONS FOR USE

RYLE'S TUBE

Materials Used

- Poly Vinyl Chloride (PVC) / Poly urethane, Polyethylene, ABS.

Indications

- Introduction of nutrition and aspiration of intestinal secretion.

Contraindications

- Not for parenteral use.
- Not to be used in patients with known hypersensitivity to any of the materials used.

Directions for Use

For Nasal Route

- Keep the patient in lying position and place a linen-savor pad across the patient chest.
- Lubricate the tip of the tube with water-soluble lubricant.
- Examine the nasal route and insert the tip of the tube through nasal route & turn the tube towards nasopharyngeal junction and advance it downward.
- Patient may be allowed to sip water to facilitate easy incubation up to the marking (Applicable only if patient is in sitting/semi erect position).
- Closer with 6% taper is provided for connection with male fitting.
- Fix the tube properly.

For Oral Route

- Keep the chin of the patient to his chest to close his trachea and open his mouth.
- Lubricate the tip of the tube with water-soluble lubricant.
- Place the tip of the tube under the tongue of the patient and give him some water to sip and advance the tube till the mark.
- Fix the tube properly.

Cautions


- Do not use if package is opened or damaged.
- The patient is allowed to speak minimum to ensure the tube is not in larynx.
- The product should be used only by qualified healthcare professionals.
- Do not re-sterilize. Discard the set after single use.
- Reuse and cleaning of product may alter their structural and mechanical properties. It may lead to infection or other illness/injury.
- The product should be replaced and disposed of as per facility approved protocol or CDC guidelines.
- Store in between 5°C to 35°C, avoid excessive heat, protect from direct sunlight and moisture.
- Poly Medcure Limited will not be responsible for any direct incidental or consequential damages resulting from reuse of product.

 **POLYMED**

Reg No. under Rule-13(2) of PWM Rules:
BO-04-000-12-AAACP3891P-24

 **POLY MEDCURE LTD.**

At: A-22 TO A-27, Bais Godam, Industrial Estate,
Jaipur, Rajasthan 302006, India
Web : www.polymedicure.com
Contact customer care executive:
☎ +91-129-4287053,
Email: customercare@polymedicure.com
Address: Same as above

 Obelis S.a
Boulevard Général Wahis 53
1030 Brussels, BELGIUM
Tel : +(32) 2. 732.59.54
E-mail : mail@obelis.net


0123



Cautions



See Instructions for use



Product reference/Art. No.



For single use only



Do not Resterilize



Do not use if packaging or product has been damaged or contaminated



Batch Number



Date of Manufacturing



Use by / Expiry date



Sterilised by Ethylene Oxide



Single Sterile Barrier System



European Authorised Representative



Manufactured by



Non Pyrogenic



Storage Condition



Keep Away from Sunlight



Keep Dry



Medical Devices

Not made with DEHP plasticizers

Rev. 06-25/08