

# INSTRUCTIONS FOR USE

## SUCTION CATHETER PLAIN

### Materials Used

- Poly Vinyl Chloride (PVC).

### Indications

- To clean the secretion from the respiratory tract.

### Contraindications

- Not to be used in patients with known hypersensitivity to any of the materials used.

### Directions for Use

- Using aseptic technique, remove the suction catheter from its protective package.
- Attach the suction catheter to the outlet port of the suction machine.
- Turn on the suction machine and check for negative pressure. To do this, kink the connecting tubing with the machine running and note the reading on the gauge. The correct setting should be (3) for adults, (2) for children and (1) for infants. Adjust the pressure by turning the adjustment knob on the suction machine.
- Insert the suction catheter into the mouth advancing slowly to the back. If patient starts to cough or gag wait until the patient recovers before continuing. Never suction for longer than 15 seconds.
- After suctioning the patient, suction water through the suction catheter until the catheter and tubing are clear. Never allow the suction container bottle to rise above the fill limit line.
- Turn machine off and wash your hands.

### Cautions


- Do not use if package is opened or damaged.
- The patient is allowed to speak minimum to ensure the tube is not in larynx.
- The product should be used only by qualified healthcare professionals.
- Do not re-sterilize. Discard the set after single use.
- Reuse and cleaning of product may alter their structural and mechanical properties. It may lead to infection or other illness/injury.
- The product should be replaced and disposed of as per facility approved protocol or CDC guidelines.
- Store in between 5°C to 35°C, avoid excessive heat, protect from direct sunlight and moisture.
- Poly Medure Limited will not be responsible for any direct incidental or consequential damages resulting from reuse of product.

 **POLYMED**

Reg No. under Rule-13(2) of PWM Rules:  
BO-04-000-12-AAACP3891P-24

 **POLY MEDURE LTD.**

At: A-22 TO A-27, Bais Godam, Industrial Estate,  
Jaipur, Rajasthan 302006, India  
Web : [www.polymedure.com](http://www.polymedure.com)  
Contact customer care executive:  
☎ +91-129-4287053,  
Email: [customercare@polymedure.com](mailto:customercare@polymedure.com)  
Address: Same as above

 Obelis S.a  
Boulevard Général Wahis 53  
1030 Brussels, BELGIUM  
Tel : +(32) 2. 732.59.54  
E-mail : [mail@obelis.net](mailto:mail@obelis.net)

  
0123



Cautions



See Instructions for use



Product reference/Art. No.



For single use only



Do not Resterilize



Do not use if packaging or product has been damaged or contaminated



Batch Number



Date of Manufacturing



Use by / Expiry date



Sterilised by Ethylene Oxide



Single Sterile Barrier System



European Authorised Representative



Manufactured by



Non Pyrogenic



Storage Condition



Keep Away from Sunlight



Keep Dry



Medical Devices

Not made with natural rubber latex  
Not made with DEHP plasticizers