

# INSTRUCTIONS FOR USE

## YANKAUR SUCTION SET

### Materials Used

- Poly Vinyl Chloride (PVC), K-Resins.

### Indications

- To clear the airway for intubation or removal of secretion.
- To clear the blood secretion.

### Contraindications

- Can damage the soft tissue during insertion.

### Directions for Use

- Using aseptic technique, fix small connector on handle and big connector on suction pump.
- Lubricate the tip of the yankaur tube with water-soluble lubricant.
- Insert tip of handle in body part and let it remove secretion by suction.
- Control the degree of the suction by applying the thumb over the hole present in the back of proximal part of yankaur.

### Warning

- Do not attempt to remove a solid object or an inhaled foreign body from the back of the throat with suction. This could result in the object being forced further into the airway and possibly causing a complete obstruction.

### Cautions

- Do not use if package is opened or damaged.
- The product should be used only by qualified healthcare professionals.
- Do not re-sterilize. Discard the set after single use.
- Reuse and cleaning of product may alter their structural and mechanical properties. It may lead to infection or other illness/injury.
- The product should be replaced and disposed of as per facility approved protocol or CDC guidelines.
- Store in between 5°C to 35°C, avoid excessive heat, protect from direct sunlight and moisture.
- Poly Medicure Limited will not be responsible for any direct incidental or consequential damages resulting from reuse of product.



**POLY MEDICURE LTD.**

At: E-971-972 & G-973-974, Biotech Park,  
Sitapura Industrial Area, Jaipur, Rajasthan 302022, India  
Web : [www.polymedicure.com](http://www.polymedicure.com)  
Contact customer care executive: ☎+91-129-4287053,  
Email: [customer care@polymedicure.com](mailto:customer care@polymedicure.com)  
Address: Same as above

Reg No. under Rule-13(2) of PWM Rules:  
BO-04-000-12-AAACP3891P-24

**EU REP** Obelis S.a  
Boulevard Général Wahis 53  
1030 Brussels, BELGIUM  
Tel : +(32) 2. 732.59.54  
E-mail : [mail@obelis.net](mailto:mail@obelis.net)



Cautions



See Instructions for use



Product reference/Art. No.



For single use only



Do not Resterilize



Do not use if packaging or product has been damaged or contaminated



Batch Number



Date of Manufacturing



Use by / Expiry date



Sterilised by Ethylene Oxide



Double Sterile Barrier System



European Authorised Representative



Manufactured by



Non Pyrogenic



Storage Condition



Keep Away from Sunlight



Keep Dry



Medical Devices

Not made with natural rubber latex  
Not made with DEHP plasticizers