

# Instructions for Use

POLYTOX™

Dialysis Fluid Filter

## General information

For general information & basic detail regarding Endotoxin Filter please refer product or carton label.

## Indication:

Endotoxin dialysis fluid filter is designed for the preparation of ultrapure fluid for use in dialysis. However there should be strict monitoring & attention to the chemical & microbiological quality of water. Concentrate & the maintenance of supply systems i.e. RO system & central delivery system must be closely monitored for the better output of the product performance. Chemical quality must comply with the standards.

However the microbial quality parameters like microbial count (CFU/mL) & endotoxin measurement (EU/mL) should be water <100 CFU/mL & <0.25EU/mL for Dialysis concentrate <1000CFU/mL & <1EU/mL.

## Warnings:

Endotoxin filter also called dialysis fluid filter can be used with Dial-Ezee Polymed Dialysis Machine. However strict attention must be given to disinfection & maintenance of all equipment used for dialysis.

Please refer to the operating instructions of the dialysis machine for further details.

## Disinfection/Cleaning

Endotoxin dialysis fluid filter must be included in all Dial-Ezee Polymed Dialysis Machine for machine cleaning & all disinfection cycles.

Please refer to the operating instruction of the dialysis machine for further details.

## Filter Integrity

Endotoxin Filter is tested for filter integrity when used with Dial- Ezee Polymed Dialysis Machine. The mandatory integrity test must be performed manually when used with other models in accordance with the relevant operating instructions.

## Dialysis Fluid Filter

Endotoxin Filter must be exchanged after 12 weeks or 100 treatments or earlier if the filter fails to pass the integrity test.

Please refer to the operating instructions for further details.

## Installation/Exchange

Endotoxin dialysis fluid filter must be installed in the appropriate area in dialysis machine. However the locking system & specially designed connectors ensures the safe connection between the dialysis fluid filter & hydraulics components of the dialysis machine. However it is important to follow correct installation disinfection process.

Please refer to the operating instructions for further details.

## Warranty

If there is any defect in the manufactured product or the manufacturer shall not be liable for any kind of misuse, improper handling of the product, non compliance for the instructions for use, any cautionary notes & for any damage incurred subsequent to the manufacturer's delivery of the Endotoxin dialysis fluid filter.

However if there is any defect in the packaged product then the packaged defective product will be replaced if the same is reported timely with details of the lot no.



**POLY MEDICURE LIMITED**

Plot No. 33-34, Sector 68, IMT,  
Faridabad, Haryana 121004, INDIA  
Web: [www.polymedicure.com](http://www.polymedicure.com)  
Contact customer care executive: ☎ +91-129-4287053  
Email: [customercare@polymedicure.com](mailto:customercare@polymedicure.com)  
Address: Same as above



**EU REP** Obelis s.a.,  
Boulevard Général Wahis 53,  
1030, Brussels, BELGIUM.  
Tel : +(32) 2. 732.59.54  
E-mail : [mail@obelis.net](mailto:mail@obelis.net)

	Cautions
	See Instructions for use
<b>REF</b>	Product reference/Art. No.
	For single use only
	Do not Sterilize
	Do not use if packaging or product has been damaged or contaminated
<b>LOT</b>	Batch Number
	Date of Manufacturing
	Use by / Expiry date
<b>STERILE R</b>	Fluid Path Sterilized using irradiation
	Single Sterile Barrier System
<b>EU REP</b>	European Authorised Representative
	Manufactured by
	Fluid Path Non Pyrogenic
	Storage Condition
	Keep Away from Sunlight
	Keep Dry
<b>MD</b>	Medical Device