

INSTRUCTIONS FOR USE

EXTENSION LINE

Materials Used

- Polypropylene, Poly Vinyl Chloride (PVC), Polycarbonate, Polyethylene, ABS.

Indications

- Infusion of I.V. fluids, parenteral nutrition and administration of other drugs and medicines .

Contraindications

- Administration of highly viscous fluids.
- Not to be used in patients with known hypersensitivity to any of the materials used.

Directions for Use

- Connect the female luer connector of extension tube to the male fitting of infusion set or syringe.
- Remove the protective cap of male luer connector and expel the air from the tube and close regulating clamp.
- Connect the male luer connector to the I.V. device and regulate the flow by adjusting regulating clamp to achieve desired flow rate.

Cautions

- Do not use if package is opened or damaged.
- The product should be used only by qualified healthcare professionals.
- Do not re-sterilize. Discard the set after single use.
- Reuse and cleaning of product may alter their structural and mechanical properties. It may lead to infection or other illness/injury.
- The product should be replaced and disposed of as per facility approved protocol or CDC guidelines.
- Store in between 5°C to 35°C, avoid excessive heat, protect from direct sunlight and moisture.
- Poly Medure Limited will not be responsible for any direct incidental or consequential damages resulting from reuse of product.



Reg No. under Rule-13(2) of PWM Rules:
BO-04-000-12-AAACP3891P-24



At: A-22 TO A-27, Bais Godam, Industrial Estate,
Jaipur, Rajasthan 302006, India
Web : www.polymedicure.com
Contact customer care executive:
☎ +91 -129-4287053,
Email: customer care@polymedicure.com
Address: Same as above

EU REP Obelis S.a
Boulevard Général Wahis 53
1030 Brussels, BELGIUM
Tel : +(32) 2. 732.59.54
E-mail : mail@obelis.net



	Cautions
	See Instructions for use
	Product reference/Art. No.
	For single use only
	Do not Resterilize
	Do not use if packaging or product has been damaged or contaminated
	Batch Number
	Date of Manufacturing
	Use by / Expiry date
	Sterilised by Ethylene Oxide
	Single Sterile Barrier System
	European Authorised Representative
	Manufactured by
	Non Pyrogenic
	Storage Condition
	Keep Away from Sunlight
	Keep Dry
	Medical Devices

Not made with natural rubber latex
Not made with DEHP plasticizers