

INSTRUCTIONS FOR USE

PERITONEAL DIALYSIS CATHETER KIT

Materials Used

- Polypropylene, Polyethylene, Poly Vinyl Chloride (PVC), Acrylonitrile Butadiene Styrene (ABS), Stainless Steel, Latex.

Indications

- To collect the post operation fluid from the patient.
- To measure the volume of fluid for daily checks.

Contraindications

- Not to be used in patients with known hypersensitivity to any of the materials used.

Directions for Use

- Disinfect the skin of the abdomen and cover with sterile cloth.
- Inject local anesthetic into the insertion site.
- The skin surface is cut a few millimeter with the scalpel and the catheter with the stylet is inserted under constant control pressure.
- As soon as the peritoneum is pierced, the stylet is withdrawn and the blunt plastic catheter is carefully pushed in further.
- When the catheter is in the correct position, it is anchored with a plaster.
- The catheter connector is then fixed to the catheter.

Cautions

- Do not use if package is opened or damaged.
- The product should be used only by qualified healthcare professionals.
- Do not re-sterilize. Discard the set after single use.
- Reuse and cleaning of product may alter their structural and mechanical properties. It may lead to infection or other illness/injury.
- The product should be replaced and disposed of as per facility approved protocol or CDC guidelines.
- Store in between 5°C to 35°C, avoid excessive heat, protect from direct sunlight and moisture.
- Poly Medicure Limited will not be responsible for any direct incidental or consequential damages resulting from reuse of product.



Cautions



See Instructions for use



Product reference/Art. No.



For single use only



Do not Resterilize



Do not use if packaging or product has been damaged or contaminated



Batch Number



Date of Manufacturing



Use by / Expiry date



Sterilised by Ethylene Oxide



Single Sterile Barrier System



European Authorised Representative



Manufactured by



Non Pyrogenic



Storage Condition



Keep Away from Sunlight



Keep Dry



Medical Devices



EU REP Obelis S.a
Boulevard Général Wahis 53
1030 Brussels, BELGIUM
Tel : +(32) 2. 732.59.54
E-mail : mail@obelis.net



Not made with DEHP plasticizers

Rev.06-25/08